

longevha

by evoskin

Activate your gene of longevity

*From Cellular Longevity
to Visible Regeneration
Aesthetic Medicine
is entering a new era.*

***A revolutionary
biorevitalizing skin booster***



01

THE REGENERATIVE SKIN MANIFESTO

For decades, skin rejuvenation has relied on volume, inflammation, and mechanical stimulation. Fill, irritate, swell, repeat. Results were visible but often temporary, biologically superficial, and poorly aligned with true tissue health. We believe this approach is outdated. Real regeneration does not begin at the surface. It begins at the cell.

THE NEW PARADIGM: SEQUENCE MATTERS

Regeneration is not a single act. It is a biological sequence. We introduce a new regenerative logic

ACTIVATE LONGEVITY

Before asking the skin to regenerate, we must restore its responsiveness.

LongevHA™ is designed to:

- Reactivate SIRT1, the longevity gene
- Restore cellular energy through NAD⁺
- Reduce oxidative and inflammatory stress
- Rehydrate and stabilize the dermal matrix
- Prepare fibroblasts to respond intelligently to signals

This is not inflammation-based stimulation. This is cellular reprogramming. You cannot regenerate exhausted cells

DELIVER REGENERATION

Once the biology is restored, regeneration can be precise, efficient, and visible.

ReversHA™ is engineered to:

- Restore intercellular communication through certified milk exosomes
- Deliver PDRN, biomimetic peptides, and amino acids directly to fibroblasts
- Stimulate Type I and Type III collagen synthesis
- Modulate pigmentation pathways through glutathione and niacinamide
- Repair texture, tone, elasticity, and skin quality at the structural level

This is not volume. This is instruction-based regeneration. Cells don't heal without instructions.

WHY THIS IS DIFFERENT

This approach is: biological, not mechanical, signal-based, not trauma-based, compatible, not competitive, with devices and surgery, foundational, not temporary. It works with lasers, RF, microneedling, PRP, and surgery because it restores the biology those tools depend on.

The future of aesthetics is not about doing more. It is about doing things in the right order.

LongevHA™ turns the cell back on. ReversHA™ tells it what to do.

This is the new regenerative standard.

The new era of skin regeneration

LONGEVHA is the first revolutionary skin booster with NAD+, resveratrol and highly concentrated HA, able to stimulate SIRT1: **the gene of longevity.**

Designed to do three things simultaneously:

1. Hydrate & Lift
2. Repair & Firm
3. Protect & Rejuvenate

HIGH MOLECULAR WEIGHT HA (40 mg)

Intense superficial hydration and temporary lifting effect

LOW MOLECULAR WEIGHT HA (40 mg)

Promotes stimulation of collagen type III, growth factors and modulates inflammatory processes

MANNITOL

Has free-radicals scavenger properties, reduces inflammation from the injections, protects and prevents injected HA from degradation

NAD+

Regulates gene expression, energy metabolism, activates sirtuins

RESVERATROL

Has an antioxidant and skin-calming properties, activates sirtuins



INDICATED FOR


Wrinkles
Dull skin
UV damaged skin
Stressed skin
Decreased skin thickness
Loss of skin elasticity and firmness



Protocol

Product Use Application Methods

- Apply the product evenly to the treatment area
- Perform microneedling using a pen device set at 0.3 mm depth to enhance transdermal absorption.
- Alternatively, a transdermal ultrasonic device can be used to enhance penetration, applying the solution directly to the skin or via a cartridge
- Gently massage until fully absorbed
- Remove any excess with a damp cotton pad
- Finish with a soothing moisturizer and broad-spectrum sunscreen (SPF 30+)

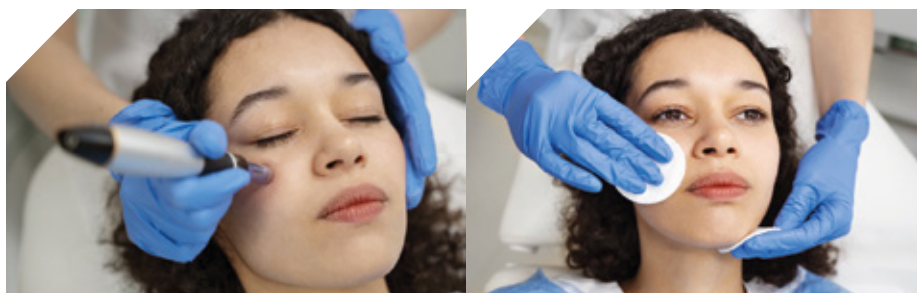

Treatment Areas

Face and Décolleté

Dosage

Face: 0.6–0.8 mL per side

Face & Décolleté: 2.5 mL total

Post-Treatment

Use a sunscreen daily for 7 days to protect the skin

Product conservation

Store in a dry, cool and clean place at a temperature between 5°C and 25°C in the original packaging.


The product has a shelf life of 3 years from the manufacturing date when stored in its original, unopened packaging under recommended conditions.

10

ingevha

*Clinical results
that exceed
expectations.*

BEFORE

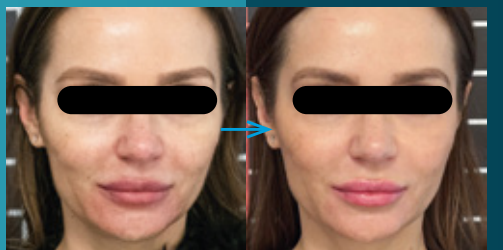

AFTER

BEFORE



AFTER

BEFORE



AFTER

BEFORE



AFTER

Technical characteristics

HYALURONIC ACID	3.2%
80 MG (40 MG HMW, 40 MG LMW)	
MANNITOL	0.5%
NAD+	
RESVERATROL	
PEPTIDES	

LONGEVHA is a revolutionary biorevitalizing skin booster with highly concentrated HA of two molecular weights, enriched with mannitol, NAD+, resveratrol and peptides.

PHARMACEUTICAL GRADE PRODUCTION

- GMP Compliant Raw Materials
- Sterile Vials
- Endotoxin Tests for each batch
- Biocompatible
- Stable and Sterile Exosomes

*Made in Italy
1 syringe of 2,5 mL
3 year shelf life*



TOPICAL USE



4300 Biscayne Blvd
Miami FL 33137
USA

T +1-802 386 7546
E info@evoskin.com
W evoskin.com

CAROMED ITALIA
MEDICAL RESEARCH